**Thickness: 2.5 mil (0,06 mm)**  
**Length: 10.0" (250 mm)**

**SafeDoff™**  
**TECHNOLOGY**

*SafeDoff™ Technology is a textured tab located on the thumb side cuff, designed for safer and faster doffing.*

POWDER-FREE • NON-STERILE • SINGLE USE • NITRILE GLOVES • SILICONE-FREE

**EN ISO 21420:2020**  
**Level 5 (Min. 1 - Max. 5)**

Dexterity	Handvaardigheid	Kätevyyks
Dextérité	Bøjlighed	Sprawność manualna
Fingerfertigkeit	Bevægelighed	Yetenekler
Destrezza	Bevegelighet	البراعة اليدوية
Destreza		

**EN ISO 374-1:2016**  
**+A1:2018/Type B**

Chemical risks	Kemiska risker
Risques chimiques	Kemiske risici
Chemische Risiken	Kjemiske risikoer
Rischi chimici	Kemialliset riskit
Riesgos químicos	Zagrożenia chemiczne
Riscos Químicos	Kimyasal Riskler
Chemische risico's	المخاطر الكيميائية

**EN ISO 374-1:2016**  
**+A1:2018**  
**Permeation**

K Sodium hydroxide 40% (1310-73-2)	Level 6
P Hydrogen peroxide 30% (7722-84-1)	Level 2
T Formaldehyde 37% (50-00-0)	Level 4

(Min. 1 - Max. 6)

Level 1 > 10 min	Level 3 > 60 min	Level 5 > 240 min
Level 2 > 30 min	Level 4 > 120 min	Level 6 > 480 min

**EN ISO 374-5:2016**

Micro-organisms risks  
Risques liés aux micro-organismes  
Risiko durch Mikroorganismen  
Rischi di microorganismi  
Riesgos de microorganismos  
Riscos de micro-organismos  
Risiko's van micro-organismen

Risker för mikroorganism  
Risici för mikroorganism  
Risikoer for mikroorganism  
Mikro-organismen riskit  
Ryzyko drobnoustrojów  
Mikroorganizma riskleri  
مخاطر الكائنات المجهرية

**Protection against bacteria and fungi - Pass**  
**Protection against viruses - Pass**

**EN ISO 374-2:2019**  
**Penetration**

Level	AQL	Inspection level
Level 3	<0.65	G1
Level 2	<1.5	G1
Level 1	<4.0	S4

**For Contact with Foodstuffs: COMMISSION REGULATION**  
**(EU) No. 10/2011, COMMISSION REGULATIONS (EC)**  
**No. 1935/2004, No. 2023/2006 and FDA 21 CFR 177.2600**

**CE 2777**

Regulation (EU) 2016/425

Module B EU Type Examination and Module C2  
Ongoing Conformity SATRA Technology Europe Ltd.  
Bracetown Business Park, Clonee, Dublin D15 YN2P,  
Ireland (Notified Body No. 2777)

SINGLE USE ONLY

NOT MADE WITH NATURAL RUBBER LATEX

KEEP DRY

KEEP AWAY FROM SUNLIGHT

**EN** Gloves have a shelf life of 3 years • Potential allergen: Carbamate • Gloves provide protection from chemical hazards shown • Before usage, inspect the gloves for any defects or imperfections • It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion, and degradation • The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only (except in cases where the glove is equal to or over 400 mm - where the cuff is tested also) and relates only to the chemical tested, it can be different if the chemical is used in a mixture • Permeation information does not reflect the actual duration of protection in the workplace and the differentiation between mixtures and pure chemicals • When used, protective gloves may provide less resistance to the dangerous chemical due to changes in physical properties • Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual use time significantly • Where the test specimens gave an increased puncture force after chemical exposure, the result is reported as a negative degradation • For corrosive chemicals, degradation can be the most important factor to consider in selection of chemical resistant gloves • The penetration resistance has been assessed under laboratory conditions and relates only to the tested specimen • Donning: Thoroughly wash hands. Select the appropriately sized gloves. Hold with one hand and insert the other. Pull glove cuff towards wrist to cover as much skin as possible and secure glove. Check to make sure there are no holes or tears. Doffing: Grasp the tab located below the thumb to pull glove away from hand, remove glove. For the remaining glove, insert index finger under the cuff and peel it off from the inside, turning it inside out to create a "bag" that contains both gloves. • All Showa products must be stored unopened at room temperature and protected from heat above 40°C (104°F), humidity, sunlight, ozone, pests and sharp objects • It is recommended that all Showa products be subject to a "First In, First Out" stock rotation. Products with or without an expiry date do not release the customer from ensuring product viability upon receipt and do not warrant the fitness of a product for any particular use • Discard used gloves in compliance with local regulations

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<b>EU DECLARATION OF CONFORMITY</b> www.showagroup.com		 <b>KNOW YOU'RE PROTECTED</b>	<b>www.chemrest.com</b> Type on the icon is in accordance with EN ISO 374-1:2016+A1:2018 standard.
 <b>Manufactured for:</b> Showa Best Glove, Inc. 579 Edison Street Menlo, GA 30731 USA 800.241.0323 USA@SHOWAGroup.com	 <b>Importer &amp; Authorised Representative</b> Showa International (Netherlands) B.V. WTC Amsterdam, Tower 2, 3rd floor Strawinskylaan 1817, 1077 XX Amsterdam The Netherlands +31 (0)88-004 2100 info@SHOWAGroup.eu		